

References

1. South S. T., et al. ACMG Standards and Guidelines for constitutional cytogenomic microarray analysis, including postnatal and prenatal applications: revision 2013. *Genetics in Medicine* **15**(11):901–909 (2013).
2. Ambros I., et al. Ultra-high density SNParray in neuroblastoma molecular diagnostics. *Frontiers in Oncology* **4**:202 (2014).
3. Zimmerman E., Maron J. L. *FOXP2* gene deletion and infant feeding difficulties: a case report. *Cold Spring Harbor Molecular Case Studies* **2**:a000547 (2016).
4. Rodriguez-Pascau L., et al. Characterization of two deletions involving NPC1 and flanking genes in Niemann-Pick type C disease patients. *Molecular Genetics and Metabolism* **107**(4):716–720 (2012).
5. Chen W., et al. Identification of chromosomal copy number variations and novel candidate loci in hereditary nonpolyposis colorectal cancer with mismatch repair proficiency. *Genomics* **102**(1):27–34 (2013).
6. Mason-Suares H. Density matters: comparison of array platforms for detection of copy number variation and copy-neutral abnormalities. *Genetics in Medicine* **15**(9):706–712 (2013).
7. Mayer A., et al. Homozygosity mapping and whole-genome sequencing reveals a deep intronic PROM1 mutation causing cone-rod dystrophy by pseudoexon activation. *European Journal of Human Genetics* **24**(3):459–462 (2016).
8. Darcy D., et al. Mosaic paternal genome-wide uniparental isodisomy with Down syndrome. *American Journal of Medical Genetics Part A* **167**(10):2463–2469 (2015).
9. Oneda B., et al. High-resolution chromosomal microarrays in prenatal diagnosis significantly increase diagnostic power. *Prenatal Diagnosis* **34**(6):525–533 (2014).
10. Jiangchuan T., et al. Concurrence of B-lymphoblastic leukemia and myeloproliferative neoplasm with copy neutral loss of heterozygosity at chromosome 1p harboring a MPL W515S mutation. *Cancer Genetics* **207**(10–12):489–494 (2014).
11. Choi S., et al. Near-haploid B lymphoblastic leukemia with an apparent hyperdiploid karyotype: the critical role of SNP analysis in establishing proper diagnosis. *Journal of Hematopathology* **7**(1):27–32 (2014).
12. Fitzgerald T. W., et al. Large-scale discovery of novel genetic causes of developmental disorders. *Nature* **519**(7542):223–228 (2015).

Ordering information

Part number	Description	Details
CytoScan® HD Suite consumables		
901835	CytoScan® HD Array Kit and Reagent Kit Bundle	Arrays and reagents sufficient for 24 reactions
N/A	Chromosome Analysis Suite (ChAS)	Available as free download from www.affymetrix.com/chas
CytoScan® training products		
901834	CytoScan® HD Training Kit	Arrays and reagents sufficient for 24 reactions plus training materials
000802	CytoScan® FAS On-Site Training	FAS-led on-site preparation and first week of training
000803	CytoScan® FAS Assisted Training	FAS-led on-site preparation. Customer completes training using self-paced tools
Supporting products		
00-0218	GeneChip® 3000 7G with Workstation and AutoLoader	Includes <ul style="list-style-type: none"> ▪ GeneChip® Scanner 3000 7G with AutoLoader ▪ n2D Handheld Barcode Reader ▪ GeneChip® Fluidics Station 450 ▪ GeneChip® Hybridization Oven 645 ▪ Computer Workstation with instrument control software
00-0334	GeneChip® System 3000Dx v.2*	Includes <ul style="list-style-type: none"> ▪ GeneChip® Scanner 3000Dx v.2 with AutoLoaderDx ▪ GeneChip® Fluidics Station 450Dx v.2 ▪ Workstation with Affymetrix Molecular Diagnostics Software (AMDS) *Requires GeneChip® Hybridization Oven 645 (see below)
00-0331	GeneChip® Hybridization Oven 645	
00-0401	Affymetrix® NIMBUS® Target Preparation Instrument	

For additional instrument system configurations or individual system components to meet your needs, please contact your local Affymetrix Account Manager.

Affymetrix, Inc: (US) +1-888-362-2447, +1-408-731-5000 ■ (EU) +44-(0)1628-552550 ■ (JP) +81-(0)3-6430-4020 ■ (CN) +86-21-63915511

eBioscience Products: (US) +1-888-999-1371, +1-858-642-2058 ■ (EU) +43 1 796 40 40 305 ■ (JP) +81-(0)3-6430-4020

USB Products: (US) +1-800-321-9322, +1-216-765-5000 ■ (EU) +44-(0)1628-552600

www.affymetrix.com Please visit our website for international distributor contact information.

For Research Use Only. Not for use in diagnostic procedures.

P/N CL00704 Rev. 6

© 2012–2016 Affymetrix, Inc. All rights reserved. Affymetrix®, Axiom®, GeneChip®, CoMAP™, Command Console®, CytoScan®, DMET™, Eureka™, Eureka Genomics®, Eureka Genotyping™, Expression Console™, GeneAtlas®, GeneChip-compatible™, GeneTitan®, Genotyping Console™, myDesign™, MyGeneChip™, NetAffx®, OncoScan®, Powered by Affymetrix™, PrimeView®, and ViewRNA® are trademarks or registered trademarks of Affymetrix, Inc. Please see affymetrix.com/trademarks for a complete list of Affymetrix trademarks. All other trademarks are the property of their respective owners.